


CELOXTM Saves Time. Saves Blood. Saves Lives.



The Best of the '3-Minute' Gauzes... ARE YOU CARRYING THE BEST?

5' Z-Fold - P/N - FG08838031
10' Z-Fold - P/N - FG08834012, NSN# 6510-01-619-8636
Also available in 10' Roll P/N - FG08834011

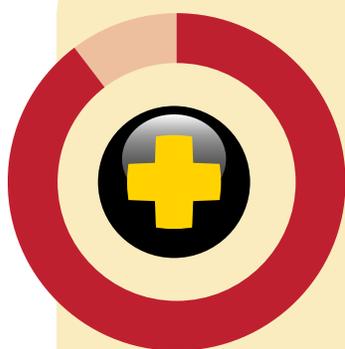

90%
SURVIVAL RATE

CeloxTM Gauze demonstrated higher rates of survival when compared to the current standard of care, Combat GauzeTM.



CHITOGAUZETM
SURVIVAL RATE



COMBATGAUZETM
SURVIVAL RATE

CeloxTM Gauze had the highest observed survival rate with 90% of the animals surviving, while Combat GauzeTM had a 60% survival rate.⁽¹⁾

CeloxTM Gauze, designed and intended for the control of severe bleeding, offers numerous Clinical, Tactical, and Economical advantages over other '3-Minute' hemostatic dressings such as Combat GauzeTM, Combat GauzeTM XL, and ChitoGauzeTM.

CLINICAL

When placed directly into a bleeding wound and subjected to pressure CeloxTM Gauze absorbs blood and forms a muco-adhesive gel that seals the wound to stop the flow of blood.

In a recent Navy study⁽¹⁾, CeloxTM Gauze clearly and consistently outperformed competitors in several areas relevant to a Combat or Tactical Medic. They include: Immediate Hemostasis (defined as a 3 minute compression period), Post-Application Blood Loss, and most notably, Survival.

TACTICAL

You don't always get to choose your Phase of Care.

'3 Minutes' of direct pressure begins after dressing application ends. 'Packs faster' can be the difference between your safety and tactical medical superiority for your casualty. CeloxTM 5' Gauze is nearly 60% shorter in length compared to competitive products while filling the same size wound. The shorter gauze allows significantly faster wound packing. CeloxTM 5' Z-Fold Gauze is intuitive, easy to handle, and reduces application time during all phases of care.

ECONOMICAL

We recognize the pressures departments and agencies face to provide greater medical capability with diminishing resources. CeloxTM hemostatic agents have been carefully designed to offer a comprehensive range of clinically and tactically superior hemorrhage control solutions, yet priced to maximize adoption and fielding.



(1) Rall JM et al. Comparison of Novel Hemostatic gauzes to Quikclot Combat Gauze in a standardized swine model of uncontrollable hemorrhage. TR-2012-22

*Combat GauzeTM is a registered TM of Z-Medica Corporation. ChitoGauzeTM is a registered TM of HemCon Medical Technologies.

CELOXTM Saves Time. Saves Blood. Saves Lives.


The *BEST* of the '3-Minute' Gauzes...

CeloxTM Gauze is a high performance hemostatic gauze that offers clinical, tactical, and economical advantages over competitive '3-Minute' Gauzes.



5' Z-Fold - P/N - FG08838031
10' Z-Fold - P/N - FG08834012, NSN# 6510-01-619-8636
Also available in 10' Roll P/N - FG08834011


Bleeding starts.

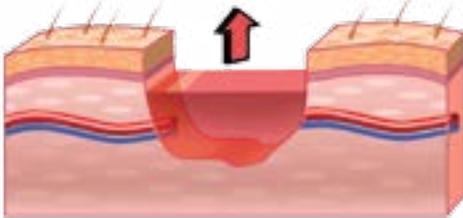


Fill wound with CeloxTM Z-Fold Gauze



Tightly pack the whole space

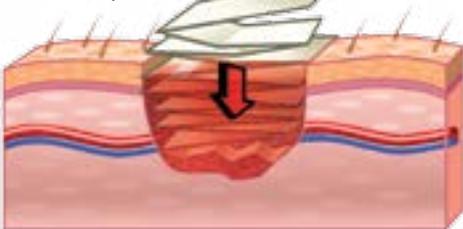

Compress

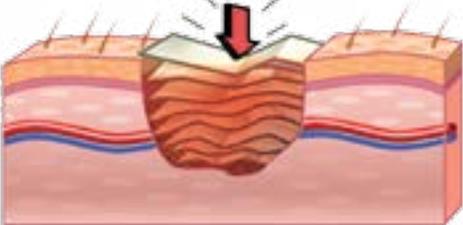

How to Use...

Instructions for use...

1. Tear open pack. Take out the CeloxTM Z-Fold Gauze and take hold of one end with the other hand.
2. Tightly pack the unfolding CeloxTM Z-Fold Gauze directly to the source of the bleeding. Pack remaining wound cavity with CeloxTM Z-Fold Gauze or standard gauze. Excess CeloxTM Z-Fold Gauze can be torn or cut if necessary.
3. Apply FIRM pressure directly to the wound for a 3 minutes. If bleeding persists apply direct pressure for an additional 3 minutes.
4. Wrap and tie with a bandage so as to maintain pressure on the wound.
5. Discard any remaining CeloxTM Z-fold Gauze.
6. Transfer patient to medical facility as soon as possible.
7. Show empty pack to medical personnel.

Contact us at www.biostatllc.com | 407.252.6758
[facebook.com/tactical.biostat](https://www.facebook.com/tactical.biostat)

